# beurer BY 15

<b>D</b>	<b>Manuelle Milchpumpe</b>		<b>I</b>	<b>Tiralatte manuale</b>	
	Gebrauchsanweisung .....	2		Istruzioni per l'uso .....	34
<b>GB</b>	<b>Manual breast pump</b>		<b>TR</b>	<b>Manüel süt pompası</b>	
	Instructions for use .....	10		Kullanım kılavuzu .....	42
<b>F</b>	<b>Tire-lait manuel</b>		<b>RUS</b>	<b>Ручной молокоотсос</b>	
	Mode d'emploi .....	18		Инструкция по применению .....	50
<b>E</b>	<b>Extractor de leche manual</b>		<b>PL</b>	<b>Ręczny laktator</b>	
	Instrucciones de uso .....	26		Instrukcja obsługi .....	59

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Lessingstraße 10 b

89231 Neu-Ulm, Germany

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

## Table of contents

<b>1. Signs and symbols</b> .....	<b>11</b>
<b>2. Getting to know your device</b> .....	<b>11</b>
<b>3. Intended use</b> .....	<b>11</b>
<b>4. Notes</b> .....	<b>12</b>
<b>5. Device description</b> .....	<b>13</b>
<b>6. Initial use</b> .....	<b>13</b>
6.1 Assembling the breast pump .....	14
6.2 Cleaning and sterilising the breast pump .....	14
<b>7. Operation</b> .....	<b>14</b>
7.1 Tips to ensure a better milk flow .....	14
7.2 Operating the breast pump .....	15
7.3 Storing the breast milk .....	15
7.4 Feeding with the breast milk .....	16
<b>8. What if there are problems?</b> .....	<b>16</b>
8.1 Replacement parts and wearing parts .....	16
<b>9. Technical data</b> .....	<b>17</b>

### Included in delivery

- Manual breast pump
- Pump attachment with silicone membrane and silicone valve
- 1x silicone cushions (standard size)
- 1x bottles
- 1x bottle caps
- Silicone teat
- Bottle cap for silicone teat
- Bottle holder
- 2x adapter for Avent and NUK bottles
- These instructions for use

## Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage, beauty, air and baby care. Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer team

## 1. Signs and symbols

The following symbols appear in these instructions for use:

	<b>WARNING</b> Warning instruction indicating a risk of injury or damage to health
	<b>IMPORTANT</b> Safety note indicating possible damage to the device/accessory
	<b>Note</b> Note on important information
	Observe the instructions for use
	Manufacturer
 Operating	Permissible operating temperature and humidity
 Storage	Permissible storage temperature and humidity
	Indicates that the packaging contains products that are intended to come into contact with food pursuant to Regulation (EC) No 1935/2004.
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on class I medical products.

## 2. Getting to know your device

The Beurer BY 15 breast pump set allows nursing mothers to express their breast milk easily and in the comfort of their own homes. If not enough breast milk is taken from mothers who have recently given birth, this can lead to the painful build-up of milk, which in the worst-case scenario could cause inflammation of the nipples (mastitis). This build-up of milk can be prevented through targeted expressing of breast milk using the breast pump.

In addition, breast pumps help to alleviate cracked or sore nipples. You can use the integrated stimulation function to draw out flat or inverted nipples, making it easier to then pump the breast milk. The breast pump also allows you to store breast milk in the bottle, for example if you will be travelling or away from your baby for an extended period of time.

Babies who cannot be breast-fed directly for health reasons (due to a premature birth or cleft palate) can still be given breast milk by using the breast pump.

## 3. Intended use

The breast pump must be used exclusively for pumping breast milk from nursing mothers. Do not use the breast pump on animals. The breast pump is only intended for domestic/private use, not for commercial use.

This device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

## 4. Notes

### **WARNING**

#### **Safety notes**

- Keep packaging material away from children to avoid danger of suffocation.
- Keep small parts out of reach of children.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- The breast pump must not come into contact with hot surfaces or sharp-edged objects.
- Do not use any additional parts that are not recommended by the manufacturer or offered as accessories.
- For hygiene reasons, the breast pump may only be used on one person. Do not share the breast pump with other nursing mothers.
- The milk pump must only be used on the female breast.
- Never use the breast pump if feeling drowsy or when asleep.
- Never use the breast pump in the bath or the shower.
- Repairs must only be carried out by Customer Services or authorised retailers. Under no circumstances should you open or repair the breast pump yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will result in voiding of the warranty.
- Never place the breast pump in the microwave.
- Never heat up breast milk in the microwave as it may be heated unevenly and could burn your child when they drink it.
- This device must not be used by people with reduced physical, sensory or mental skills or a lack of experience or knowledge.
- The breast pump does not require calibration
- The breast pump and accessories must not be modified.

### **IMPORTANT**

#### **Instructions for transporting the bottle**

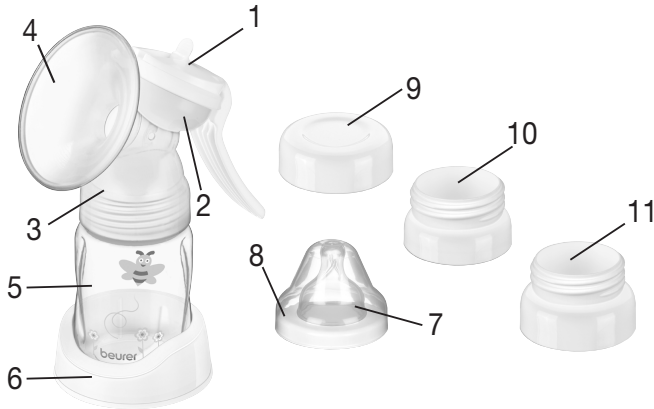
- When transporting breast milk in the bottle (e.g. when on the move or travelling), make sure that the bottle cap is always screwed on tightly.

### **WARNING**

#### **Child safety**

- This device may only be used under adult supervision.
- Feeding from the teat continuously and for long periods can lead to tooth decay.
- Use the teat for feeding, not as a pacifier.
- Always check the temperature of the milk before feeding.
- Inspect the device before each use and pull the teat in all directions.
- Dispose of the teat at the first sign of damage or wear and tear.
- Do not expose the teat to direct sunlight or sources of heat and do not leave in disinfectant ("sterilising solution") for longer than recommended as this may damage the teat.
- Keep the teat away from sweet substances as these may cause tooth decay.
- Do not use any abrasives or anti-bacterial cleaning agents.
- Sterilise all parts that come into contact with the breast milk for five minutes in boiling water.
- Clean the device before each use.
- Make sure that the temperature of the liquid in the bottle never exceeds 50°C.

## 5. Device description

1. Manual pump handle (2 levels)	7. Teat
2. Silicone membrane	8. Bottle cap for teat
3. Silicone valve	9. Screwed plug for bottle
4. Silicone cushion (standard size)	10. Adapter for NUK (the breast pump can also be used with this bottle if desired)
5. Bottle (180 ml)	11. Adapter for Avent (the breast pump can also be used with this bottle if desired)
6. Bottle holder	

## 6. Initial use

### Note

Check that the packaging of the breast pump has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the breast pump or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

## 6.1 Assembling the breast pump

Once the breast pump has been removed from the packaging, check the plastic containers in particular for cracks.

<p>1. Screw the pump attachment onto the bottle. Ensure the pump attachment is firmly secured so that no liquid can escape.</p>	
<p>2. Attach the handle for the manual pump including the silicone membrane by pressing the silicone membrane firmly onto the pump attachment until it clicks into place.</p>	

## 6.2 Cleaning and sterilising the breast pump

### WARNING

**Clean and sterilise the separate parts of the breast pump that come into direct contact with the breast milk individually before first use and any further use.**

To sterilise the breast pump correctly, proceed as follows:

1. Unscrew the pump attachment from the bottle.
2. Remove the silicone membrane and silicone valve from the pump attachment.
3. Sterilise the bottle and pump attachment (with silicone membrane and silicone valve) for five minutes in boiling water or using a steam steriliser. Do not use the breast pump until all individual parts are completely dry. Do not use any antibacterial cleaning products, since these may damage the plastic on the breast pump.

## 7. Operation

### 7.1 Tips to ensure a better milk flow

- Choose a location to use the pump where you can relax.
- Choose a time to use the pump when you can relax.
- Having your child nearby or looking at a picture of your child can have a positive effect on the milk flow.
- If feeding your child using one breast, use the breast pump on the other breast at the same time.
- Warmth and relaxation encourage milk flow. Use the breast pump after having a shower or bath.
- If your breasts are swollen or sore, place a warm towel on them for a few minutes. This will improve the milk flow.
- If the milk does not start to flow from your breasts immediately, try to relax and then try again a few minutes later.
- If the breast pump still does not extract any milk from your breasts after five minutes, stop using it and try again later.
- If you experience any sharp pains when using the breast pump, consult a doctor.

## 7.2 Operating the breast pump

<p>1. Before using the breast pump, wash your hands and breasts thoroughly.</p>	
<p>2. Sit with your upper body lent forwards slightly. You can support your back better by using a pillow.</p>	
<p>3. Then place the breast shield with silicone cushion gently on the breast. Make sure that the nipple is completely enclosed and there are no air pockets. To ensure proper function, the breast shield must be drawn firmly onto the breast. Use the manual pump as shown in the image.</p>	
<p>4. You can vary the pump intensity by selecting between the MIN/ MAX markings that can be seen. We recommend starting in the MIN position.</p>	

## 7.3 Storing the breast milk

Expressed milk can be stored in the fridge or freezer for a certain period of time.

- When storing breast milk, ensure that the breast pump and the bottles are disinfected before usage. Only store breast milk in disinfected containers.
- Store breast milk in the fridge or freezer immediately after expressing. Do not leave breast milk out at room temperature.
- If storing breast milk in the freezer, it is recommended that you mark the bottle with the date of expressing.

### **WARNING**

Observe the storage periods in the following table:

	ROOM TEMPERATURE	REFRIGERATOR (approx. 3 - 5°C)	FREEZER (approx. -16°C)
Recently expressed breast milk	Can be kept for 6 hours	Can be kept for 24 hours (Do not store in the fridge door, as it is not sufficiently cool there)	Can be kept for 3 months
Defrosted breast milk (that had been frozen)	Use immediately	10 hours	Do not refreeze

## 7.4 Feeding with the breast milk

### Note

You can use the supplied bottle with the silicone teat when feeding with the breast milk. Alternatively, you can also express the breast milk with an NUK or Avent bottle before feeding (corresponding NUK / Avent adapter is included in delivery).

- Prior to feeding, ensure the bottle has been disinfected before filling it with the breast milk that has been stored.
- To defrost breast milk, store it overnight in the fridge, so as to retain the nutrients. Once the breast milk is defrosted, it must be used within 24 hours. Breast milk should only be defrosted in warm water in especially urgent cases.
- If the breast milk smells unpleasant, it should be disposed of immediately.
- Never heat up breast milk in the microwave as it may be heated unevenly and could burn your child when they drink it.
- If heating up the breast milk, check its temperature before feeding it to your baby.

## 8. What if there are problems?

Problem	Potential solutions
There is no suction from the breast pump.	<ul style="list-style-type: none"><li>• Check all parts of the breast pump are correctly assembled.</li><li>• Check whether the pump attachment is correctly placed on the breast.</li></ul>
No breast milk is being pumped.	<ul style="list-style-type: none"><li>• Ensure that there is no milk remaining in the silicone valve and obstructing the milk flow.</li></ul>
Pumping the breast milk is causing pain.	<ul style="list-style-type: none"><li>• Try to relax and then attempt it again a few minutes later. As you use the breast pump more and more, you will find it easier to pump milk.</li><li>• Set the suction level to MIN.</li><li>• Consult your doctor.</li></ul>
The breast milk is leaking out underneath the silicone cushion.	<ul style="list-style-type: none"><li>• Remove the silicone cushion and re-attach it. Ensure the silicone cushion is correctly positioned and firmly secured to the pump attachment.</li><li>• Lean your upper body forwards slightly.</li></ul>
One of the breast pump components has become altered or damaged.	<ul style="list-style-type: none"><li>• Do not use any corrosive, gaseous cleaning agents or solvents.</li><li>• In some cases, marked fluctuations in temperature can damage the plastic on the breast pump.</li><li>• If any of the parts becomes damaged, stop using the breast pump.</li></ul>

### 8.1 Replacement parts and wearing parts

<u>Item</u>	<u>Item number</u>
Bottle with teat and bottle cap	163.719
Adapter for Avent and NUK bottles	163.720
2 silicone cushions (small and standard)	163.721
Manual pump attachment including silicone cushion (standard size)	163.722
Bottle stand	163.723

## 9. Technical data

Model	BY 15
Dimensions	200 x 160 x 66 mm
Weight	135 g
Maximum suction pressure	-0.32 bar
Permissible operating conditions	+5°C to +40°C, 30% to 85% relative humidity (non-condensing), 700 – 1060 hPa ambient pressure
Permissible storage conditions	-20°C to +60°C, 10% to 95% relative humidity (non-condensing), 700 – 1060 hPa ambient pressure
Safety requirements according to EN 60601-1	
Child use and care articles - Drinking equipment according to EN 14350	
Device for expressing breast milk	
Device with fine vacuum and low flow	

Technical information is subject to change without notification to allow for updates

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